

Linearity FD General Chemistry Panel 2 for Abbott Systems

REF

K941M-5

10 x 3 mL

LOT

07161

Aalto Scientific Ltd
230 Technology Parkway
Eatonton, GA 31024
USA



2027-11-17

INTENDED USE

The Linearity FD General Chemistry Panel 2 for Abbott Systems is assayed quality control material consisting of two sets of five levels of human serum. Each level of Linearity FD General Chemistry Panel 2 for Abbott Systems Set 1 contains the following analytes: Albumin, ALP, Amylase, AST, Activated AST, CK, GGT, LDH, Lipase, Total Protein. Each level of Linearity FD General Chemistry Panel 2 for Abbott Systems Set 2 contains the following analytes: ALT, Activated ALT. These five levels of each set demonstrate a linear relationship to each other for their respective analytes¹. It is intended to simulate human patient serum samples for purpose of determining linearity, calibration verification and verification of reportable range for Albumin, ALP, Amylase, AST, Activated AST, CK, GGT, LDH, Lipase, Total Protein and ALT, Activated ALT.

This product is intended for use with quantitative assays on the indicated analyzer provided in the labeling. The Linearity FD General Chemistry Panel 2 for Abbott Systems should not be used for calibration or standardization of the Albumin, ALP, Amylase, AST, Activated AST, CK, GGT, LDH, Lipase, Total Protein and ALT, Activated ALT assays. The Linearity FD General Chemistry Panel 2 for Abbott Systems is for In Vitro Diagnostic use only.

SUMMARY AND PRINCIPLE

As defined in the Clinical Laboratory Improvement Amendments of 1988 (CLIA) by the Centers for Medicare and Medicaid Services (CMS) and the Centers for Disease Control (CDC), each laboratory must revalidate each test method's AMR at least every six months as well as following changes in lots of analytically critical reagents or major system components². Good laboratory practices require that stable reference materials be used to verify the accuracy and precision of testing methods and techniques. Linearity FD General Chemistry Panel 2 for Abbott Systems may be used as one would use human whole blood to verify and validate the AMR.

WARNINGS AND PRECAUTIONS

Because this product is of human origin, it should be handled as though capable of transmitting infectious diseases. Each serum, plasma or whole blood donor unit used in the preparation of this material was tested by United States Food and Drug Administration (FDA) approved methods and found to be negative for antibodies to HIV and HCV and nonreactive for HBsAg. Because no test method can offer complete assurance that HIV, hepatitis B virus, and hepatitis C virus or other infectious agents are absent, this material should be handled as though capable of transmitting infectious diseases. This product may also contain other human source material for which there is no approved test. The FDA recommends such samples be handled at the Centers for Disease Control's Biosafety Level 2.

Linearity FD General Chemistry Panel 2 for Abbott Systems is intended solely for the purpose of in vitro diagnostic use as described on the label. Audit® MicroControls™, Inc. will not be liable for any unclaimed damages arising from any other usage.

MATERIALS PROVIDED

Linearity FD General Chemistry Panel 2 for Abbott Systems Set 1, 5 X 3mL

Linearity FD General Chemistry Panel 2 for Abbott Systems Set 2, 5 X 3mL

STORAGE AND STABILITY

Linearity FD General Chemistry Panel 2 for Abbott Systems is stored at 2-8°C and will remain stable in the unopened vial until the expiration date. After opening, the contents should be used according to the instrument manufacturer's instructions and immediately returned to 2-8°C.

When used to monitor the precision of laboratory testing procedures for its assays, Linearity FD General Chemistry Panel 2 for Abbott Systems has a reconstituted stability of up to 5 days under the proper storage conditions. Leaving the vial uncapped, or prolonging its time at room temperature, will void this open vial stability claim. Make sure the contents of the vial are well mixed before use.

PROCEDURE

Follow the manufacturer's instructions provided for quality control and for verifying and validating the AMR. Verify that the lot number on each vial matches the package insert. To avoid evaporation, do not leave the vial uncapped. Q.C. requirements should be performed in conformance with local, state and/or federal regulations or accreditation requirements. Calibration verification linearity material should be run³:

- every six (6) months.
- when a complete change of reagents for a procedure is introduced.
- when there is major preventive maintenance or replacement of critical parts that may influence test performance.
- when control materials reflect an unusual trend or shift, or are outside of the laboratory's acceptable limits.
- when the laboratory's established schedule for verifying the reportable range for patient test results requires more frequent calibration verification.

INSTRUCTIONS FOR USE

- Remove a vial from the package and reconstitute with 3 mL of deionized water.
- Allow the vial to sit at room temperature for 5 minutes.
- Occasionally swirl for 45 minutes, or until all visible material is dissolved. Do not shake. Do not mix mechanically. Avoid getting any undissolved material on the sides of the vial or the stopper.

- When all visible solid material is dissolved, invert several times to dissolve any material on the stopper.
- Swirl occasionally for at least 5 minutes.
- Use immediately or return to 2-8°C.
- The vial should remain stored at 2-8°C at all times. If additional sampling is necessary, the time outside of 2-8°C storage should be minimized.

CALCULATIONS OF RESULTS

Each set of Linearity FD General Chemistry Panel 2 for Abbott Systems is prepared in a manner such that an equal distance exists between each consecutive level. This dilution scheme is consistent with the CLSI recommendation¹ for preparing linearity sets.

U.S. customers only - Once each vial of the total set is tested, raw data may be entered via the AUDITOR™ QC Program at www.auditmicro.com. An on-line graph showing actual values versus predicted values for each analyte is then available to print, along with slope and intercept data. Call (866) 25-AUDIT for more information.

LIMITATIONS OF THE PROCEDURE

If the contents of any of the vials become frozen, discard all vials and request a replacement set, as the results will not be valid.

Dispose of any discarded materials in accordance with the requirements of your local waste management authorities.

EXPECTED VALUES

Each lot of product is manufactured such that a linear relationship exists among levels. Actual results obtained may vary depending on instrumentation, methodology and assay temperature. Results may also be dependent on the accuracy of the instrument/reagent system calibration. The degree of acceptable non-linearity is an individual judgment based on methodology, clinical significance and medical decision levels of the test analyte. The material and information presented here in no manner constitutes an overruling of any federal, state or other regulatory body's regulations and/or guidelines.

ORDERING INFORMATION

PRODUCT NUMBER	PRODUCT DESCRIPTION	PRODUCT PACKAGING
K941M-5	Linearity FD General Chemistry Panel 2 for Abbott Systems	10 x 3 mL

Distributed by Audit® MicroControls™, Inc. - U.S. customers only please call (866) 252-8348 or www.auditmicro.com

¹ Dilution schemes are based on guidelines provided by CLSI in approved guideline EP6-A, "Evaluation of the Linearity of Quantitative Measurement Procedures: A Statistical Approach; Approved Guideline", April 2003.

² Federal Register 42 CFR Part 493, Department of Health and Human Services, January 24, 2003; p.3690.

³ Federal Register 42 CFR Part 493, Department of Health and Human Services, January 24, 2003; §493.1255, (b) (1) (ii).

Set 1							
	Units	Reagent, Instrument	A	B	C	D	E
ALBG	g/dL	Abbott Alinity	1.8	4.0	6.0	7.9	9.0
ALBP	g/dL	Abbott Alinity	1.6	3.8	5.7	7.6	8.7
ALP	IU/L	Abbott Alinity	10	1362	2600	3858	>4555
AMY	IU/L	Abbott Alinity	10	1664	3140	4671	5701
AST	IU/L	Abbott Alinity	10	1069	2010	2929	3522
Activated AST	IU/L	Abbott Alinity	8	988	1847	*	*
CK	IU/L	Abbott Alinity	11	1201	2302	3399	4193
GGT	IU/L	Abbott Alinity	11	2206	4175	6165	7615
LDH	IU/L	Abbott Alinity	<30	1159	2174	3200	3887
LIP	IU/L	Abbott Alinity	9	246	>300	>300	>300
TP	g/dL	Abbott Alinity	2.3	5.6	8.8	12.0	14.1

*Unable to obtain value

Set 2							
	Units	Reagent, Instrument	A	B	C	D	E
ALT	IU/L	Abbott Alinity	17	926	1759	2608	>3258
Activated ALT	IU/L	Abbott Alinity	8.2	1014	>1200	>1200	>1200



Catalog Number



For In Vitro
Diagnostic Use



Use By
(YYYY-MM-DD)



Lot Number



Caution



www.auditmicro.com/inserts

2 – 8°C



Temperature Limit



Manufactured By



Reconstitute With

3.0 mL
DI H₂O